Rotaid AED Cabinets
SOLID PLUS

INSTALLATIEHANDLEIDING • GUIDE D'INSTALLATION • MONTAGEANLEITUNG

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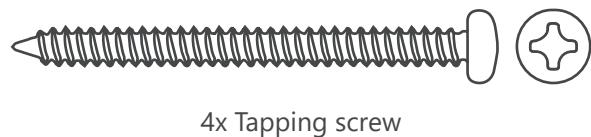
Parts-list

This Rotaid package contains:

- Installation manual
- Polycarbonate backplate
- Polycarbonate cover
- Alarm (if ordered)
- Fastening materials

*Screw Set:

Index

| | | |
|----------------|--|--------------------------|
| pag. 4 | | English instructions |
| pag. 5 | | Nederlandse instructies |
| pag. 6 | | Instructions en Français |
| pag. 7 | | Deutsche Anleitung |
| pag. 8 | | Instrucciones en Español |
| pag. 9 | | Istruzioni Italiano |
| pag. 10 | | Polskie instrukcje |
| pag. 11 | | User manual |

Rotaid contact information

Keizersveld 52D
5803 AN
Venray
The Netherlands

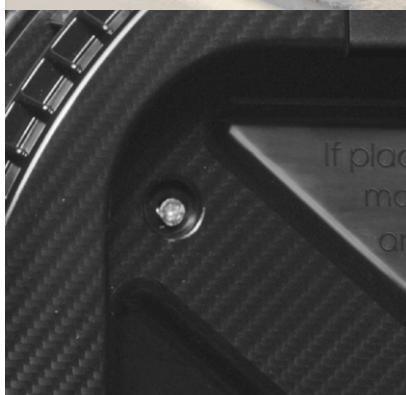
www.rotaid.com
 info@rotaid.com
 +31 (0)478 769 052

ROTAID SOLID PLUS Installation manual

1. Determine the location where you wish to mount the Rotaid. Make sure there are no irregularities present at the desired location. Please take the fire safety and evacuation regulations into account. If mounted on a stone or concrete wall, please use the wall anchors prior to fixation.

2. Mark the 4 holes for drilling and use the screws and rings for fixation.

3. Use an Allen key (5mm) to firmly attach the nylon belt to the back plate.

4. After attaching the belt and cover, now please mount both long bolts in the indicated holes. Mount these by hand via a screwdriver and do not overtighten. You can now hang the AED via its handle on the bolts.

5. You can now place the AED into the cabinet by using its handle or carrying case handle. Please rotate the cover on the back plate by starting slightly counterwise and rotate to the right until the cover reaches the stop. When correctly placed, the hook on the bottom of the cabinet falls through the cover.

Your Rotaid Plus is now ready for use!

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